

<b>Klas Therapy Protocol</b>				
Disease / Condition	Therapy Time (Minutes)		Frequency	Dosage
	Klas-M50 (3cm <sup>2</sup> )	Klas-L20 (2cm <sup>2</sup> )	Time/Week	(J/cm <sup>2</sup> )
<b>Acne Vulgaris</b>				
Facial acne	10	14	2~3	8
Neckline acne	12	17	2~3	10
Back acne	15	21	2~3	12
<b>Aesthetic</b>				
Age spot	15	21	2~3	12
Cholasma	15	21	2~3	12
Facial anti-aging: General skin conditioning	10	14	1~2	8
Scar: fresh	15	21	2~3	12
Stretch mark (Melasma): fresh	15	21	2~3	12
Wrinkle and fine lines	10	14	2~3	8
<b>Pain Relief</b>				
Achill bursitis	10	14	3~7	8
Acute & Chronic Joint Pain; Finger	7	11	3~7	6
Acute & Chronic Joint Pain; Toe	7	11	3~7	6
Acute & Chronic Joint Pain; Wrist	12	17	3~7	10
Acute & Chronic Joint Pain; Elbow	12	17	3~7	10
Calcaneal Spur	12	17	3~7	10
Carpal Tunnel Syndrome (CTS)	14	21	3~7	12
Herpes Simplex (HSV1)	4	6	2/Day	3
Pain associated with Facial Palsy	10	14	3~7	8
Plantar Fasciitis- Heel Pain	14	21	3~7	12
Post Operative Tissue Pain	10	14	1~2/Day	8
<b>Respiratory Tract and Ear, Nose, Throat</b>				
Rhinitis	10	14	3~7	8
Sinusitis	7	11	3~7	6
Tinnitus	30	45	3~7	25
<b>Skin Diseases</b>				
Allergic rashes	4	5	2~3	3
Aphthous ulcers & sore gums	4	5	2~3	3
Burn & related wounds	6	9	1~3/day	5
Cellulitis	12	17	3~7	8~10
Couperose (mild)	4	5	2~3	3
Dermatitis, allergic	4	5	2~3	3
Dermatitis, atopic	4	5	2~3	3
Dermatitis, contact	4	5	2~3	3
Eczema	4	5	2~3	3
Hypertrophic cicatrix	12	17	3	8~10
Insect bites	4	5	1~2/day	3
Pressure sore	4	5	3~7	3
Psoriasis	12	17	2~3	8~10
Rosacea	6	9	3~7	5
Skin eruption	4	5	2~3	3
Sunburn	4	5	2~3	3
Tinea pedis	12	17	2~3	8~10
Vitiligo (Leukoderma)	12	17	2~3	8~10
Warts, verruca	12	17	3	8~10
Wound, operation wound	12	17	1~3/day	8~10
Wound & ulcers	6	9	1~3/day	5
<b>Other Symptoms and Prophylaxis</b>				
Hemorrhoid	14	21	1/Day	12
Gingival and periodontal disease	4	6	1/Day	3
Oral ulcer	7	11	1/Day	6
Edema	14	21	1/Day	12
Erectile Dysfunction	30	45	2	25
<b>Therapeutic windows range from typically <math>\pm 50\%</math> of given values.</b>				