



# Therapy Guide: Hair Loss



Laser Hood: *Megalas-HR-400A*

40 x 660nm / 100mW



Guide-1	<b>By Classification - Recommended</b>				
	<b>Application</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	<b>Norwood Hamilton Classifications</b>	<b>Ludwing Classifications</b>			
	NW2	Type 1	15	2 ~ 3	7.2
	NW3, NW4	Type 2, Type 3	20		9.8
NW5	Type 4, Type 5	30	14.4		
Guide-2	<b>By Balding Area</b>				
	<b>Balding Classification</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	Balding		13 ~ 25	2 ~ 3	6 ~ 12
	Thinning		6		3
Normal		2	1		
Guide-3	<b>By Conditions</b>				
	<b>Condition</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	Androgenetic Alopecia		15 ~ 30	2 ~ 3	7.2 ~ 14.4
	Alopecia Areata				
	Before/After Hair Transplantation		20	2	9.8
Hair Loss Prevention		15	1	7.2	



# Therapy Guide: Hair Loss



Laser Hood: *Megalas-HR-400B*

30 x 660nm/100mW + 10 x 780nm/100mW



Guide-1	<b>By Classification - <i>Recommeded</i></b>				
	<b>Application</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	<b>Norwood Hamilton Classifications</b>	<b>Ludwing Classifications</b>			
	NW2	Type 1	15	2 ~ 3	7.2
	NW3, NW4	Type 2, Type 3	20		9.8
NW5	Type 4, Type 5	30	14.4		
Guide-2	<b>By Balding Area</b>				
	<b>Balding Classification</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	Balding		13 ~ 25	2 ~ 3	6 ~ 12
	Thinning		6		3
Normal		2	1		
Guide-3	<b>By Conditions</b>				
	<b>Condition</b>		<b>Therapy Time (Minutes)</b>	<b>Frequency (Time/Week)</b>	<b>Recommended Dose (J/cm<sup>2</sup>)</b>
	Androgenetic Alopecia		15 ~ 30	2 ~ 3	7.2 ~ 14.4
	Alopecia Areata				
	Before/After Hair Tranplantation		20	2	9.8
Hair Loss Prevention		15	1	7.2	