






Therapy Guide: Body Contouring

Laser Irradiation System: Megalas-AM-800



Therapy Time (Minutes)	Laser Dose (J/cm ²) in Treatment Area = 314 cm ²			Frequency (Time/Week)
	 0.0032 J/cm ² -sec 1 irradiator	 0.0064 J/cm ² -sec 2 irradiators (to same area)	 0.0096 J/cm ² -sec 3 irradiators (to same area)	
	40	7.7	15.4	
30	5.8	11.5	17.3	
20	3.8	7.7	11.5	
15	2.9	5.8	8.6	

■ J/cm²-sec = Unit Energy Density (Height=30 cm, Area=314 cm²)


The laser dose is calculated with the application of the quantity of irradiators to the treatment area .

The therapy time is decided by patient's condition and areas. **30 minutes is recommended.**

An exercise program follows the laser therapy is recommended.



Therapy Guide: Hair Loss

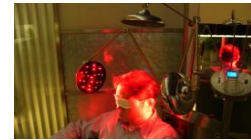

Laser Irradiation System: Megalas-AM-800-666



Guide-1: By Classification - **Recommended**

Application		Therapy Time (Minutes)	Frequency (Time/Week)	Recommended Dose (J/cm ²)
Norwood Hamilton Classifications	Ludwing Classifications			
NW2	Type 1	15	2 ~ 3	2.9
NW3, NW4	Type 2, Type 3	20		3.8
NW5	Type 4, Type 5	30		5.8

Guide-2: By Balding Area

Balding Classification	Therapy Time (Minutes)	Frequency (Time/Week)	Recommended Dose (J/cm ²)
Balding	16 ~ 32	2 ~ 3	3 ~ 6
Thinning	8		1.5
Normal	3		0.5